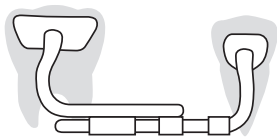
# EZ Space Maintainer™

Developed by Dr. Güray

## Space Maintaining and Space Regaining Instructions

L'espace maintenant et l'espace regagnant des instructions  
Espacio que mantiene y espacio que recupera instrucciones  
Spazio che effettuano e spazio che riguadagna le istruzioni  
Raum, der beibehalten und Raum, der Anweisungen wiedergewinnt

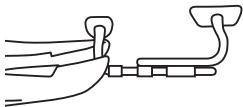
17401 Commerce Park Blvd. Tampa, FL 33647 USA

(813) 991-5896 • Fax: (813) 991-5986


[www.ezspacemaintainer.com](http://www.ezspacemaintainer.com)

# Space Maintaining Instructions


1


2

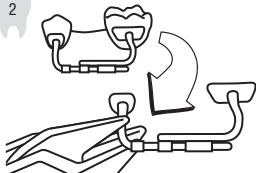
3

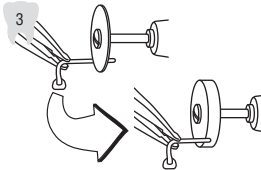4

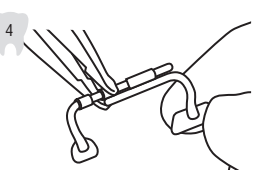5
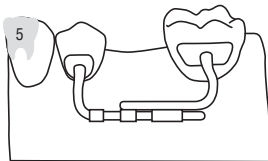
### **[English] Instructions for Space Maintaining**

Choose the proper space maintainer by its color mark and remove the coil spring.

**Green** - upper right

**Yellow** - upper left

**Red** - lower right

**White** - lower left

1. While holding the space maintainer with PANTERA® Standard Utility Pliers, adjust the length of the space maintainer.
2. Check occlusally to ensure the bases adapt perfectly to the tooth surfaces. If necessary, additional adaptation could be performed with a PANTERA® Three Prong Plier.
3. If shortest setting is needed for a small space, the space maintainer rod will need to be trimmed and polished for patient comfort. (see example)
4. Secure the length and orientation of the space maintainer by squeezing the stopper with PANTERA® Standard How Utility Pliers.
5. Etch each tooth surface for one minute. Rinse and dry thoroughly. Bond retainer with Resilience® Light-Cure Adhesive and light cure each tooth.

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

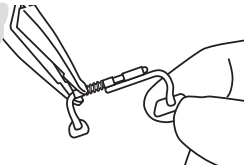
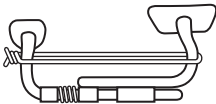
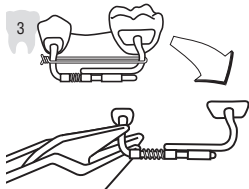
4

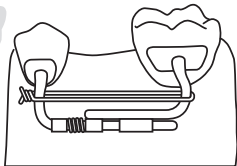5
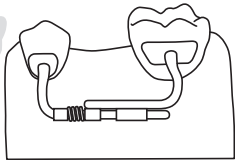
## **[English] Instructions for Space Regaining**

Choose the proper space maintainer by its color quadrant mark.

**Green** - upper right

**Yellow** - upper left

**Red** - lower right

**White** - lower left

1. Before activating the coil spring, squeeze the stopper with PANTERA® Standard How Utility Pliers.
2. Adjust the length of the space maintainer by applying a depressive force with stainless steel ligature wire using a Mathieu Hemostat. Slowly twist the stainless steel ligature wire to the necessary length.
3. Using a PANTERA® Standard How Utility Plier, check occlusally to be sure that the bases adapt perfectly to the tooth surfaces. If necessary, additional adaptation can be performed with a PANTERA® Three Prong Plier.
4. Etch each tooth surface for one minute. Rinse and dry thoroughly. Bond the space maintainer with Resilience® Light-Cure Adhesive and light cure each tooth.
5. Cut the stainless steel ligature wire. The space maintainer will start to apply force to regain the lost space or up-right the adjacent teeth. (Pay attention to the Rules of Anchorage).

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**WARNING:** This product contains nickel and should not be used for individuals with known allergic sensitivity to nickel.

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**CAUTION:** U.S. Federal law restricts this device to sale by or on the order of a health care professional.

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Emergo Europe, Molenstraat 15,  
2513 BH, The Hague, The Netherlands

© 2009 Ortho Technology, Inc. EZ Space Maintainer is a trademark, PANTERA and Resilience are registered trademarks of Ortho Technology, Inc. All rights reserved worldwide. Worldwide patents pending.

PC SKU #: L-00233 REV-0