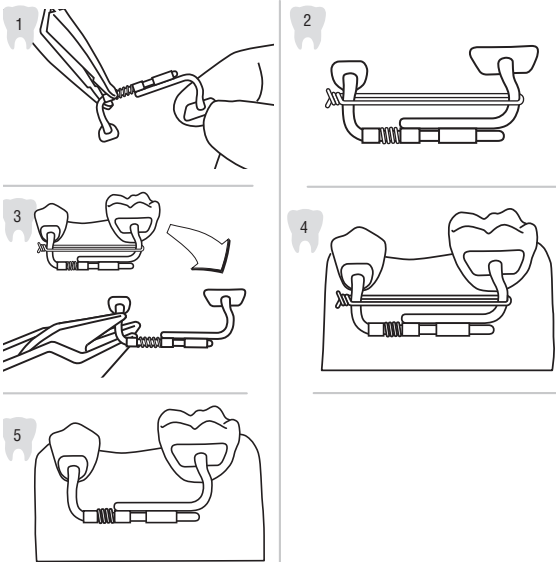
Space Regaining Instructions

[English] Instructions for Space Regaining

Choose the proper space maintainer by its color quadrant mark.

Green - upper right **Yellow** - upper left
Red - lower right **White** - lower left

- Before activating the coil spring, squeeze the stopper with PANTERA® Standard How Utility Pliers.
- Adjust the length of the space maintainer by applying a depressive force with stainless steel ligature wire using a Mathieu Hemostat. Slowly twist the stainless steel ligature wire to the necessary length.
- Using a PANTERA® Standard How Utility Plier, check occlusally to be sure that the bases adapt perfectly to the tooth surfaces. If necessary, additional adaptation can be performed with a PANTERA® Three Prong Plier.
- Etch each tooth surface for one minute. Rinse and dry thoroughly. Bond the space maintainer with Resilience® Light-Cure Adhesive and light cure each tooth.
- Cut the stainless steel ligature wire. The space maintainer will start to apply force to regain the lost space or up-right the adjacent teeth. (Pay attention to the Rules of Anchorage).

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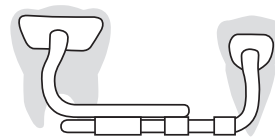
EZ Space Maintainer®

Developed by Dr. Güray

Space Maintaining and Space Regaining Instructions

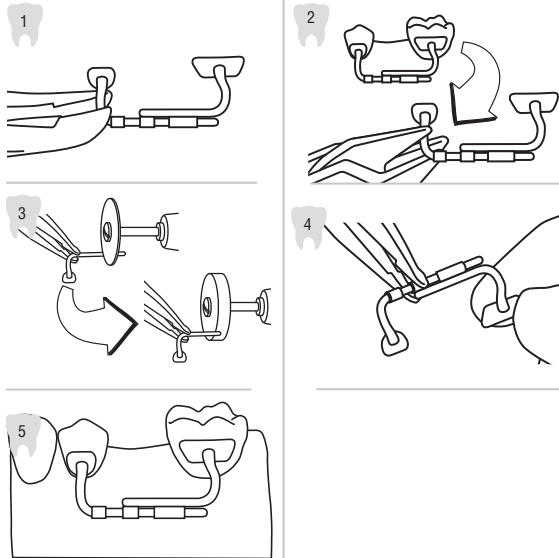
L'espace maintenant et l'espace regagnant des instructions
 Espacio que mantiene y espacio que recupera instrucciones
 Spazio che effettuano e spazio che riguadagna le istruzioni
 Raum, der beibehalten und Raum, der Anweisungen wiedergewinnt

4614 Pet Lane, Suite D-101 • Lutz, FL 33559 USA
 US: 1-800-999-3161 • Outside US: 1-813-501-1650

www.ezspacemaintainer.com

Space Maintaining Instructions

[English] Instructions for Space Maintaining

Choose the proper space maintainer by its color mark and remove the coil spring.

Green - upper right

Yellow - upper left

Red - lower right

White - lower left

1. While holding the space maintainer with PANTERA® Standard Utility Pliers, adjust the length of the space maintainer.
2. Check occlusally to ensure the bases adapt perfectly to the tooth surfaces. If necessary, additional adaptation could be performed with a PANTERA® Three Prong Plier.
3. If shortest setting is needed for a small space, the space maintainer rod will need to be trimmed and polished for patient comfort. (see example)
4. Secure the length and orientation of the space maintainer by squeezing the stopper with PANTERA® Standard How Utility Pliers.
5. Etch each tooth surface for one minute. Rinse and dry thoroughly. Bond retainer with Resilience® Light-Cure Adhesive and light cure each tooth.

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WARNING: This product contains nickel and should not be used for individuals with known allergic sensitivity to nickel.

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CAUTION: U.S. Federal law restricts this device to sale by or on the order of a health care professional.

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Emergo Europe, Prinsessegracht 20, 2514 AP, The Hague, The Netherlands

© 2017 Ortho Technology, Inc. EZ Space Maintainer, PANTERA and Resilience are registered trademarks of Ortho Technology, Inc. All rights reserved worldwide. Worldwide patents pending.