

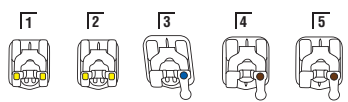
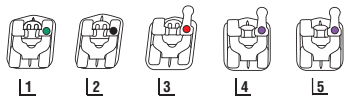
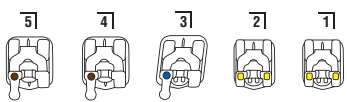
Ortho Technology's version of the Roth R[®]
"Does not imply doctor endorsement."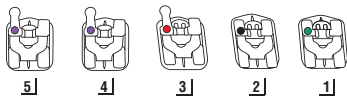

PC SKU #: L-00655 REV-0
© 2013 Ortho Technology, Inc.
Sensation is a trademark of Ortho Technology.

Instructions Enclosed

**Opening Method —
Sensation M
SLB Opening
Instrument
OT-8920**

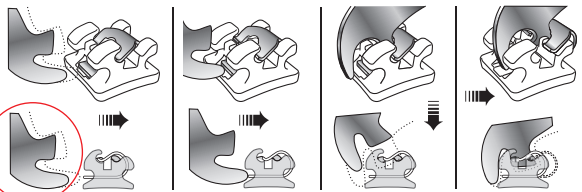Sensation™ M Active Self-Ligating Bracket System

Instructions For Use

Opening

To open the Sensation M Active Self-Ligating Bracket spring clip, place instrument on the open clip slot directly beneath the bracket interface and gently push in the occlusal direction to disengage the bracket clip. Then place the instrument directly in the facial opening of the clip to engage the small extension of the spring clip. Pull in the occlusal direction. The spring clip will pop open.

Closing

To close the Sensation M Active Self-Ligating Bracket spring clip, simply put light finger pressure on the occlusal surface of the bracket. Make sure the spring clip fully closes so the archwire is securely engaged.

Warning

If the bracket and/or clip should become damaged or broken during use, it is recommended to replace the bracket.

Sensation™ M

Active Stainless Steel Self-Ligating
Bracket System

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