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  Emurgo Europe, Prinsessegracht 20,
2514 AP, The Hague, The Netherlands

Plier Sterilization

Warning: This product is NOT sterilized. If this product is to be used intraorally, it must be sterilized by an approved method prior to use.

Ortho Technology's instruments were tested and found to be corrosion resistant when subjected to the following sterilization methods and the stated procedures were strictly followed.

Autoclave: National Sterile Quick 9000D

1. Instrument must be thoroughly cleaned! Clean 5 minutes in an Ultrasonic General Purpose Cleaner.
2. Dry joint thoroughly with compressed air or towel dry ensuring joint is free from moisture.
3. Place instruments flat on Autoclave tray with jaws open. Use distilled water in sterilizer.
4. Twenty minute cycle in the Autoclave (260°F and 25 psi.) When equipment has cooled sufficiently, remove the instrument. Towel dry ensuring joint is free from moisture. Lubricate instruments (joint) using Silicone Lubricant Spray.

Note: Instruments left in Autoclave may show signs of corrosion if not removed when cycle is complete.

Chemiclave: Harvey 4000/Chemiclave

1. Instrument must be thoroughly cleaned! Clean 5 minutes in an Ultrasonic General Purpose Cleaner.
2. Dry joint thoroughly with compressed air or towel dry ensuring joint is free from moisture.
3. Place instrument flat onto Chemiclave tray. Place a layer of paper towels between instruments. Leave jaw open. Use distilled water in sterilizer.
4. Twenty minute cycle in the Chemiclave (250°F and 25 psi.) When cycle is complete depressurize the equipment. Allow instruments to cool sufficiently. Lubricate instruments (joint) using Silicone Lubricant Spray.

Note: Instruments left in Chemiclave may show signs of corrosion if not removed when cycle is complete.

Dry Heat: Dentrionx DDS 5000

1. Instrument must be thoroughly cleaned! Clean 5 minutes in an Ultrasonic General Purpose Cleaner.
2. Dry joint thoroughly with compressed air or towel dry ensuring joint is free from moisture.
3. Place instruments onto Dentrionx racks. Begin 20 minute cycle. Upon completion of cycle, remove instruments. Lubricate instruments (joint) using Silicone Lubricant Spray.

Cox Rapid Heat Transfer Sterilizer

1. Instrument must be thoroughly cleaned! Clean 5 minutes in an Ultrasonic General Purpose Cleaner.
2. Dry joint thoroughly with compressed air or towel dry ensuring joint is free from moisture.
3. Place instruments into Cox sterilization chamber. Begin 6 minute cycle. Upon completion of cycle remove instruments and place to be cooled. Lubricate instruments (joint) using silicone Lubricant Spray.

Cold Sterilization: 2% Glutaraldehyde

1. Instrument must be thoroughly cleaned! Clean 5 minutes in an Ultrasonic General Purpose Cleaner.
2. Dry joint thoroughly with compressed air or towel dry ensuring joint is free from moisture.
3. Immerse instrument in sterilization liquid for at least one (1) hour.
4. Remove instruments from solution. Rinse in sterile water. Towel dry ensuring joint is free from moisture.

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